

 Let's Protect™





---

# ACTISORB® SILVER 220

ANTIMICROBIAL BINDING DRESSING

STERILE R

 Systagenix

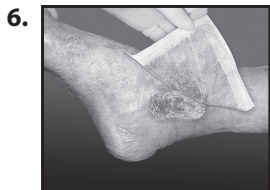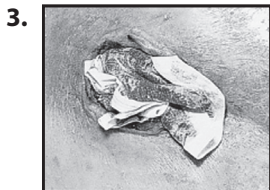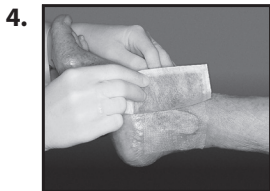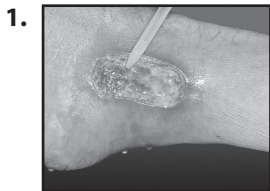

## **Product Description**

ACTISORB® Silver 220 Antimicrobial Binding Dressing is a sterile primary dressing, comprised of activated charcoal cloth, impregnated with silver, within a spun bonded nylon envelope. Within the dressing, there is 220mg silver per 100g activated charcoal cloth equating to 33µg of silver per square centimeter of cloth.

## **Indications**

ACTISORB® Silver 220 Dressing provides an effective barrier to bacterial penetration and for adsorbing offending odor resulting from wounds; the binding properties of the dressing trap bacteria, bacterial toxins and odor. ACTISORB® Silver 220 Dressing may help reduce infection in partial and full-thickness wounds, including:

- pressure ulcers
- venous ulcers
- diabetic ulcers
- first and second-degree burns
- donor sites
- surgical wounds

ACTISORB® Silver 220 Dressing is suitable for use under compression bandaging.

## **Contraindications**

ACTISORB® Silver 220 Dressing should not be used on third-degree burns.

## **Warnings**

- Should not be used on patients with a known sensitivity to silver.
- For external use only.

## **Precautions**

- This product should not be cut, otherwise particles of activated charcoal may get into the wound and cause discoloration.
- The dressing performance may be impaired by excess use of petrolatum based ointments.
- Clinicians/Healthcare Professionals should be aware that there are very limited data on prolonged and repeated use of silver containing dressings, particularly in children and neonates.

## **DIRECTIONS FOR USE**

### **Site Preparation**

Prepare wound bed according to wound management protocol, ensuring removal of necrotic tissue.

## **Dressing Application**

- The properties of the dressing are most fully utilized when ACTISORB® Silver 220 Antimicrobial Binding Dressing makes direct contact with the wound (see photo 2).
- Either side of ACTISORB® Silver 220 Dressing can be applied to the wound bed.
- Do not cut dressing. Fold or pack as necessary.
- ACTISORB® Silver 220 Dressing can be easily packed into deep wounds (see photo 3).
- The outer nylon envelope enables removal from the majority of wounds with minimal adherence or trauma.
- For lightly exuding wounds, ACTISORB® Silver 220 Dressing can be used in combination with a non-adherent wound contact layer (see photo 4).
- Depending on the level of exudate, an absorbent secondary dressing may be placed on top of ACTISORB® Silver 220 Dressing (see photo 5).
- Secure the dressing in a manner appropriate for the indication.

## **Dressing Change and Removal**

ACTISORB® Silver 220 Antimicrobial Binding Dressing can remain in situ up to 7 days, dependent on the level of exudate, while the secondary absorbent dressing is changed as required. Initially it may be necessary to change ACTISORB® Silver 220 Dressing every 24 hours (see photo 6) to ensure optimal dressing performance.

## **STORAGE INFORMATION**

Store at room temperature.

## **HOW SUPPLIED**

ACTISORB® Silver 220 Dressing

- is supplied sterile
- package contains a single dressing
- should not be cut

**Do not re-use.**

**Do not re-sterilize.**

**Do not use if package is damaged.**

**The use by date of this product is printed on the packaging.**

**Caution:** Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner.

*This caution is not applicable outside the U.S.*

## SYMBOLS USED ON LABELLING



= Do not use if package is damaged.



= Do not re-use.



= Do not re-sterilize.



= Sterile. Method of sterilization: irradiation.



= Consult instructions for use.




= Batch number.



= Use by: year and month.



= Manufacturer.

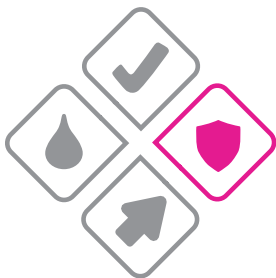

**LET'S HEAL™**

518044S.e



Manufacturer

©2011 Systagenix Wound Management Limited,  
Gargrave, North Yorkshire, BD23 3RX, U.K.

