

Designed specifically for
the **ambulatory patient**





The ACTIV.A.C.[™] Therapy Unit is portable Negative Pressure Wound Therapy designed for the ambulatory patient

The features of this ergonomically designed therapy unit include:

FOR THE PATIENT

- Lightweight. Weighs only 2.4 pounds
- Small size with a low profile that can be worn close to the body
- Easy-to-use, single-touch therapy on/off operation
- Alarm notifications that are easy to recognize and correct
- Easy, quick release 300ml canister
- 14-hour battery for activities of daily living

FOR THE CLINICIAN

SENSAT.R.A.C.[™] Technology

Only KCI V.A.C.[®] Therapy Devices provide patented SENSAT.R.A.C.[™] Technology, a real time pressure feedback system.

- Adjusts pump output to the wound site, compensating for wound distance, wound position, exudate characteristics, and patient movement
- Delivers & maintain prescribed negative pressure for optimal healing outcomes

SEAL CHECK[™] Feature

Designed to help clinicians and patients identify and troubleshoot negative pressure leaks.

- Instant Feedback. Audio and visual cues are delivered in real-time, allowing easy location of dressing leaks
- Enhanced Therapy Confidence. A proper seal helps provide accurate delivery of prescribed therapy for optimal wound healing
- Possible Reduction of Unscheduled Visits. Easier for clinicians to help troubleshoot problems over the phone

Settings Guide

Recommended therapy settings and pressure ranges are pre-programmed for indicated wound types:

- Easy Set Up. With pre-programmed settings by wound type, clinicians can initiate therapy quickly and efficiently
- Ease of Use. Intuitive control functions make it easier to use, especially for clinicians who are not as familiar with V.A.C.[®] Therapy

Therapy History Reporting

Document Patient Compliance with V.A.C.[®] Therapy.

- Monitors therapy usage, settings, durations, and dressing and canister changes to help ensure patients are receiving the prescribed course of therapy
- USB port for convenient data downloads.

Simplified Touch Screen

Enhanced, simplified control functions:

- Easy Navigation. With full-color interface with intuitive touch screen controls
- Simplified Patient Mode. Designed to be easy to use and understand
- Easier Training. With common control functions with the INFOV.A.C.[™] Therapy System



V.A.C.® Therapy Dressing Options with SENSAT.R.A.C.™ Technology



Specially designed dressings, because no two wounds are alike.



V.A.C.® SIMPLACE™ Dressing Family

Designed to simplify the V.A.C.® Therapy Dressing placement process

- Helps to promote healing by facilitating granulation tissue formation
- Fewer steps, faster application
- Spiral cut foam is simple to size; no scissors necessary
- Design allows for easier bridging



V.A.C.® GRANUFOAM™ Bridge Dressing Family

Ideal for use with wounds in pressure sensitive areas that requiring bridging

- Preassembled components specifically designed to simplify dressing application
- Integrated bridge allows for SENSAT.R.A.C.™ Pad placement away from the wound site
- Wicking layers help ensure intact skin stays dry
- Allows V.A.C.® Therapy to be used in conjunction with off-loading therapies



V.A.C.® GRANUFOAM™ Dressing

An advanced wound dressing to assist granulation tissue formation and enhance exudate removal

- Can be easily trimmed to fit the contours of deep or irregularly shaped wounds
- Can be customized for bridging techniques when treating multiple wounds



V.A.C. WHITEFOAM™ Dressing

A comfortable and versatile wound dressing

- May be used over split thickness skin grafts
- Helps promote flap and graft take
- Higher tensile strength (than V.A.C.® GRANUFOAM™ Dressing) allows for easy removal from tunnels and undermining
- Recommended in situations where hypergranulation responses are likely



SENSAT.R.A.C.™ Pad

Only SENSAT.R.A.C.™ Technology actively measures, monitors, and manages the accurate delivery of the physician's prescribed therapy to the wound site.

- Designed with patient comfort in mind
- Flexible pad material for easy application over body contours
- Discreet, low profile design

Ordering Information

Part Number Description Case Quantity

V.A.C.® Dressings with SENSAT.R.A.C.™ Pad

Standard

M8275051/10	V.A.C.® GRANUFOAM™ Small Dressing	10
M8275051/5	V.A.C.® GRANUFOAM™ Small Dressing	5
M8275052/10	V.A.C.® GRANUFOAM™ Medium Dressing	10
M8275052/5	V.A.C.® GRANUFOAM™ Medium Dressing	5
M8275053/10	V.A.C.® GRANUFOAM™ Large Dressing	10
M8275053/5	V.A.C.® GRANUFOAM™ Large Dressing	5
M8275065/5	V.A.C.® GRANUFOAM™ X-Large Dressing	5

Ease of Use

M8275041/10	V.A.C.® SIMPLACE™ Small Dressing	10
M8275041/5	V.A.C.® SIMPLACE™ Small Dressing	5
M8275040/10	V.A.C.® SIMPLACE™ Medium Dressing	10
M8275040/5	V.A.C.® SIMPLACE™ Medium Dressing	.5
M8275045/5	V.A.C.® SIMPLACE™ EX Medium Dressing	5
M8275046/5	V.A.C.® SIMPLACE™ EX Small Dressing	5
M8275042/10	V.A.C.® GRANUFOAM™ Bridge Dressing	10
M8275042/5	V.A.C.® GRANUFOAM™ Bridge Dressing	5
M8275044/5	V.A.C.® GRANUFOAM™ Bridge XG Dressing	5

Less-Adherent*

M6275033/10	V.A.C. WHITEFOAM™ Small Dressing, Foam Only	10
M6275034/10	V.A.C. WHITEFOAM™ Large Dressing, Foam Only	10
M8275067/10	V.A.C. WHITEFOAM™ Large Dressing	10
M8275067/5	V.A.C. WHITEFOAM™ Large Dressing	5
M8275068/10	V.A.C. WHITEFOAM™ Small Dressing	10
M8275068/5	V.A.C. WHITEFOAM™ Small Dressing	5

*Than V.A.C.® GRANUFOAM™ Dressings

Accessories

M6275009/10	V.A.C.® Drape	10
M6275066/10	V.A.C.® Y-Connector	10
M6275066/5	V.A.C.® Y-Connector	5
M6275069/10	V.A.C.® Tubing Cap	10
M6275069/5	V.A.C.® Tubing Cap	5

ACTIV.A.C.™ Canisters

M8275058/10 300ml	ACTIV.A.C.™ Canister w/Gel	10
M8275058/5 300ml	ACTIV.A.C.™ Canister w/Gel	5



KCI V.A.C.® Therapy has been awarded the APMA Seal of Acceptance, which recognizes products that have been found to promote quality foot health and be of significant value when used in regular professional treatment.

For more information about V.A.C.® Therapy, please call **800-275-4524** or visit **acelity.com**

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for KCI products and therapies. Please consult a physician and product instructions for use prior to application. Rx only.

Note: Disposable components of the V.A.C.® (Vacuum Assisted Closure®) Therapy System, including the foam dressing (i.e., V.A.C.® GRANUFOAM™, V.A.C.® GRANUFOAM SILVER™, or V.A.C. WHITEFOAM™ Dressing), tubing and drape are packaged sterile and are manufactured without natural rubber latex. V.A.C.® Therapy Unit canisters are packaged fluid path sterile and are manufactured without natural rubber latex. All disposable components of the V.A.C.® Therapy System are for single use only. To help ensure safe and effective use, the V.A.C.® GRANUFOAM™, V.A.C.® GRANUFOAM SILVER™ and V.A.C. WHITEFOAM™ Dressings are to be used only with V.A.C.® Therapy Units.

©2016 KCI Licensing, Inc. All rights reserved. All trademarks designated herein, unless otherwise noted, are proprietary to KCI Licensing, Inc. DSL#16-0360.US • LIT29-A-176 • (Rev. 4/16)