



V.A.C.ULTA™ Therapy Unit

INDICATIONS FOR USE

The V.A.C.ULTA™ 4 Therapy System is an integrated wound management system that provides Negative Pressure Wound Therapy (V.A.C.® Therapy) with an instillation option (V.A.C. VERAFL0™ Therapy)

- V.A.C.® Therapy in the absence of instillation is intended to create an environment that promotes wound healing by secondary or tertiary (delayed primary) intention by preparing the wound bed for closure, reducing edema, promoting granulation tissue formation and perfusion, and by removing exudate and infectious material
- V.A.C. VERAFL0™ Therapy is indicated for patients who would benefit from vacuum assisted drainage and controlled delivery of topical wound treatment solutions and suspensions over the wound bed

The V.A.C.ULTA™ 4 Therapy System with and without instillation is indicated for patients with chronic, acute, traumatic, sub-acute and dehisced wounds, partial-thickness burns, ulcers (such as diabetic, pressure and venous insufficiency), flaps and grafts

V.A.C.®Therapy in the absence of instillation may also be used for:

- The temporary bridging of abdominal wall openings where primary closure is not possible and/or repeat abdominal entries are necessary and for open abdominal wounds with exposed viscera including, but not limited to, abdominal compartment syndrome. The intended care setting is a closely monitored area, within the acute care hospital, such as the ICU. The abdominal dressing will most often be applied in the operating theater.
- The management of the environment of surgical incisions that continue to drain following sutured or stapled closure by maintaining a closed environment, and removing exudate via the application of negative pressure wound therapy.



CONTRAINDICATIONS

V.A.C.ULTA™ 4 Therapy System Contraindications

- Do not place foam dressings of the V.A.C.ULTA™ 4 Therapy System (including both V.A.C.® Therapy and V.A.C. VERAFLOR™ Therapy Dressings directly in contact with exposed blood vessels, anastomotic sites, organs or nerves*
- V.A.C.® Therapy and V.A.C. VERAFLOR™ Therapy are contraindicated for patients with:
 - » Malignancy in the wound
 - » Untreated osteomyelitis*
 - » Non-enteric and unexplored fistulas
 - » Necrotic tissue with eschar present (NOTE: After debridement of necrotic tissue and complete removal of eschar, V.A.C.® Therapy may be used)
 - » Sensitivity to silver (V.A.C.® GRANUFOAM SILVER™ Dressing and PREVENA™ Incision Dressings only)

Contraindications Specific to V.A.C. VERAFLOR™ Therapy

- Do not use V.A.C.® Dressings with Octenisept®**, hydrogen peroxide or solutions that are alcohol-based or contain alcohol
- Do not deliver fluids to the thoracic or abdominal cavity due to the potential risk to alter core body temperature and the potential for fluid retention within the cavity
- Do not use V.A.C. VERAFLOR™ Therapy unless the wound has been thoroughly explored due to the potential for inadvertent instillation of topical wound solutions to adjacent body cavities

Transitioning V.A.C.® Therapy into Home Care

- The V.A.C.ULTA™ 4 Therapy System is not intended for home use
- If there is a need to continue V.A.C.® Therapy when a patient transitions home, consider using one of the KCI Therapy Systems approved for the post-acute environment, such as:
 - » PREVENA™ 125 Therapy Unit***
 - » PREVENA PLUS™ 125 Therapy Unit***
 - » ACTIV.A.C.™ Therapy Unit
 - » V.A.C. FREEDOM™ Therapy Unit
 - » V.A.C. SIMPLICITY™ Therapy Unit
 - » V.A.C.VIA™ Negative Pressure WoundTherapy System


*Refer to Warnings section in user manual for additional information concerning bleeding and Osteomyelitis

**Not available in the United States. Brand name referenced is not a trademark of KCI, its affiliates and / or licensors.

***PREVENA™ Therapy for use on closed incisions only.

Initiating V.A.C. VERAFL0™ Therapy

Negative Pressure Wound Therapy with automated instillation and removal of topical wound solutions.

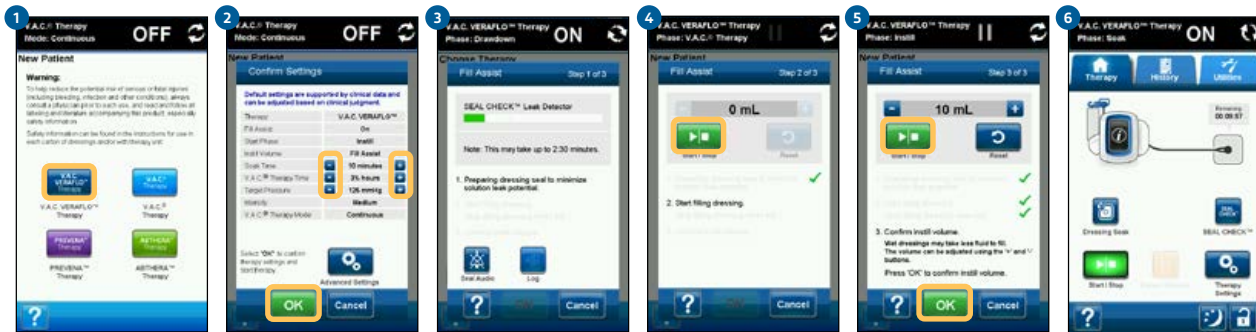

Select **V.A.C. VERAFL0™** Therapy to configure the therapy unit for use with V.A.C. VERAFL0™ Therapy

On the **Confirm Settings** screen, adjustments can be made by pressing the **+/-** buttons.

*If no changes are desired, select **OK** to Confirm Default Settings*

Fill Assist will begin to draw down the dressing to prepare the dressing seal to minimize solution leak potential.

You no longer have to wait for the green button – unit will automatically transition to the next step when the therapy unit is satisfied with the dressing leak or after 2:30 min.


Select **Start/Stop** on the Fill Assist screen to begin delivering solution to the wound. Monitor the wound as it fills with solution.

Stop Fill Assist by depressing **Start/Stop**. Select **OK** when satisfied with instillation volume

Use +/- to make minor adjustments that will be stored in memory

Initiating V.A.C.® Therapy

Negative Pressure Wound Therapy designed to deliver the wound healing benefits of V.A.C.® Therapy with patented SENSAT.R.A.C.™ Technology.

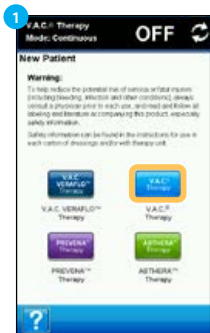

Select **V.A.C.® Therapy** to configure the therapy unit for use with V.A.C.® Therapy



On the **Confirm Settings** screen, adjustments can be made by pressing the **+/-** buttons.

*If no changes are desired, select **OK** to Confirm Default Settings*




Seal Check Leak Detector Screen—Drawdown begins



Home Screen – V.A.C.® Therapy

Initiating PREVENA™ Therapy


Negative Pressure Therapy designed specifically for the management of closed surgical incisions.

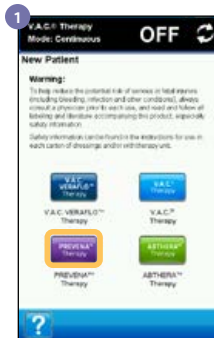

Select **PREVENA™** Therapy to configure the therapy unit for use with PREVENA™ Therapy



On the **Confirm Settings** screen, select OK to accept default settings.

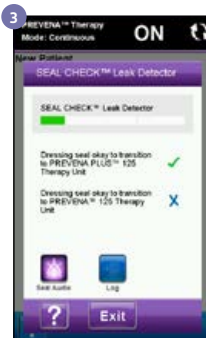


Seal Check Leak Detector Screen—Drawdown begins




Home Screen—PREVENA™ Therapy

Initiating ABTHERA™ Open Abdomen Negative Pressure Therapy

Negative Pressure Therapy designed to actively manage the open abdomen and help achieve primary fascial closure.

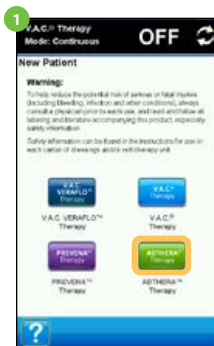

Select **ABTHERA™** Therapy to configure the therapy unit for use with ABTHERA™ Therapy

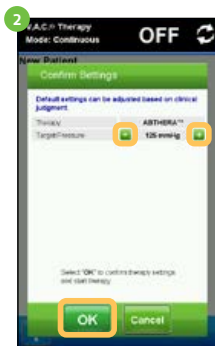

On the **Confirm Settings** screen, adjustments can be made by pressing the +/- buttons.

*If no changes are desired, select **OK** to Confirm Default Settings*



Seal Check Leak Detector Screen—Drawdown begins



Home Screen—ABTHERA™ Therapy

Insert Rep Card

Contact your Acelity Representative with questions regarding your V.A.C.ULTA™ Therapy System

Important: This diagram is provided as a basic overview only. Refer to the V.A.C.ULTA™ Therapy System User Manual and the V.A.C.ULTA™ Safety Information Sheet for detailed instructions, including important safety information. For medical emergencies, consult treating physician or call emergency services. Carefully read and follow all literature and labeling accompanying product. Rx only.

Copyright 2017 KCI Licensing, Inc. All rights reserved. All trademarks designated herein are proprietary to KCI Licensing, Inc., its affiliates and/or licensors. PRA000231-R0-US, EN • LIT#29-B-265 (Rev. 1/17)

